



# HemoPoint® H2 Optics Cleaner

## Intended Use

This cleaner is intended for cleaning the optical system of the Alere **HemoPoint® H2** Meter.


## Description

The Alere **HemoPoint® H2** Optics Cleaner consists of a spatula partially covered by a foam cushion impregnated with a special cleaning solution.

## Storage and Stability

The Optics Cleaner is stable until the expiration date, when stored unopened at 59-86°F (15-30°C). After opening, the cleaner must be used within 10 minutes.


## Directions for Use

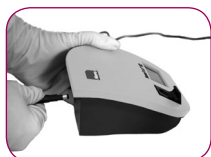

To safely clean the Alere **HemoPoint® H2** Meter optical unit, disconnect the power adaptor from the electrical connection before proceeding.



With the Alere **HemoPoint® H2** Meter in the ready mode, open the Alere **HemoPoint® H2** Microcuvette holder until you feel a resistance and the holder will not extend further.

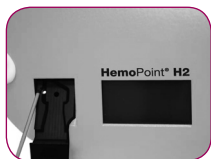

Locate the silver pin on the left-hand side of the Alere **HemoPoint® H2** Microcuvette holder.



While depressing the silver pin with a pointed object, slide the Alere **HemoPoint® H2** Microcuvette holder towards you and away from the Alere **HemoPoint® H2** Meter.



Remove the cleaner from the foil pouch. Check to be sure the pad is moist with cleaner.



Insert the cleaner slowly into the opening of the Alere **HemoPoint® H2** Microcuvette holder until you feel a smooth resistance. Slowly push the cleaner deeper into the opening until it stops.



Move the Alere **HemoPoint® H2** Optics Cleaner in an in-and-out motion for at least 5 times.



Remove the cleaner from the Alere **HemoPoint® H2** Meter. If the used cleaner is discolored or dirty, repeat the procedure using a new cleaner.



Dispose of all used cleaners as potentially infectious waste in accordance with the current regulations applicable to your establishment.



Clean and dry the Alere **HemoPoint® H2** Microcuvette holder (see the user manual, maintenance section). Wait at least 15 minutes after cleaning to re-insert the Alere **HemoPoint® H2** Microcuvette holder into the Alere **HemoPoint® H2** Meter.



Replace the Alere **HemoPoint® H2** Microcuvette holder then re-open the Alere **HemoPoint® H2** Microcuvette holder and ensure that the silver pin is locked in place.

The Alere **HemoPoint® H2** Meter is now ready to resume operation.

## Contact Alere

### Alere™ Technical Support

Contact one of the following Alere™ Technical Support Care Centers or your local distributor if you have any questions regarding the use of your Alere™ product. You may also contact us at [www.alere.com](http://www.alere.com).

| Region | Phone              | E Mail Address                 |
|--------|--------------------|--------------------------------|
| US     | + (1) 866.216.0073 | HemoPoint.H2.Support@alere.com |

### Alere™ Customer Service

Contact the following Alere™ Service Care Center or your local distributor for order and billing assistance. You may also contact us at [www.alere.com](http://www.alere.com).

#### Phone

+ (1) 877.441.7440

#### E Mail Address

clientservices@alere.com

Limited Warranty. FOR THE APPLICABLE WARRANTY PERIOD, ALERE WARRANTS THAT EACH PRODUCT SHALL BE (I) OF GOOD QUALITY AND FREE OF MATERIAL DEFECTS, (II) FUNCTION IN ACCORDANCE WITH THE MATERIAL SPECIFICATIONS REFERENCED IN THE PRODUCT MANUAL, AND (III) APPROVED BY THE PROPER GOVERNMENTAL AGENCIES REQUIRED FOR THE SALE OF PRODUCTS FOR THEIR INTENDED USE (the "LIMITED WARRANTY"). IF THE PRODUCT FAILS TO MEET THE REQUIREMENTS OF THE LIMITED WARRANTY, THEN AS CUSTOMER'S SOLE REMEDY, ALERE SHALL EITHER REPAIR OR REPLACE, AT ALERE'S DISCRETION, THE PRODUCT. EXCEPT FOR THE LIMITED WARRANTY STATED IN THIS SECTION, ALERE DISCLAIMS ANY AND ALL WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO, ANY WARRANTY OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NON-INFRINGEMENT REGARDING THE PRODUCT. ALERE'S MAXIMUM LIABILITY WITH ANY CUSTOMER CLAIM SHALL NOT EXCEED THE NET PRODUCT PRICE PAID BY CUSTOMER. NEITHER PARTY SHALL BE LIABLE TO THE OTHER PARTY FOR SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, INCLUDING, WITHOUT LIMITATION, LOSS OF BUSINESS, PROFITS, DATA OR REVENUE, EVEN IF A PARTY RECEIVES NOTICE IN ADVANCE THAT THESE KINDS OF DAMAGES MIGHT RESULT.

The Limited Warranty above shall not apply if the Customer has subjected the Product to physical abuse, misuse, abnormal use, use inconsistent with the Product Manual or Insert, fraud, tampering, unusual physical stress, negligence or accidents. Any warranty claim by Customer pursuant to the Limited Warranty shall be made in writing within the applicable Limited Warranty period.

Distributed by:

**Alere North America, Inc.**  
30 South Keller Road, Suite 100  
Orlando, FL 32810  
[www.alere.com](http://www.alere.com)

Made in Germany.

The Alere Logo and Alere are trademarks of the Alere group of companies.  
HemoPoint is a trademark of Stanbio Laboratory L.P. under license.

© 2011 Alere. All rights reserved. PN: 23003 Rev.A 2011/11